

OPTIMAL TREATMENT WORKFLOW

LONG CALCIFIED LESION



OPERATORS

Emanuele Barbato MD, PhD
Carlos Collet MD, PhD

SITE

Catheterisation Laboratory Cardiovascular
Center OLV Aalst, Belgium

PATIENT CHARACTERISTICS

- 70 year-old male
- Arterial hypertension, dyslipidemia, diabetes mellitus and COPD
- Atypical chest pain
- Diagnosis: angiography shows long calcified disease in proximal and LAD

PRECISION PCI PLAN WITH FULL PHYSIOLOGY AND IMAGING GUIDED PCI

FFR Pre-PCI functional evaluation

Ascertainment of lesion significance. Severity of ischemia.

OCT Pre-PCI imaging evaluation

(Ultrareon™ 1.0 Software)

Plaque characterization.
Selection of landing zones and stent length.
Assessment of vessel diameter.

OCT Post-PCI imaging evaluation

Stent expansion, apposition, and necessity of further optimization. Evaluation of stent edges for dissection and residual stenosis.

FFR + Pullback Post-PCI functional evaluation

Functional result of PCI. Assessing residual focal pressure losses requiring additional treatment. Prognostic stratification.


PROCEDURE

Pre-PCI Functional Assessment




Pressure measurement of the LAD: FFR pullback identified mid-segment focal functional disease and some proximal diffuse disease.

OCT Assessment

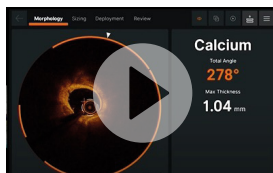


Pre-PCI assessment with Ultrareon™ 1.0 Software co-registration: after rotational atherectomy performed, the MLD-MAX algorithm is used to assess vessel morphology and plan lesion preparation and stent sizing.

Stent Deployment

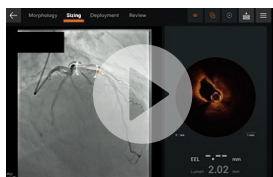


Two stent strategy deployed in the LAD: first stent (XIENCE™ 3.0x48mm) landed exactly where the co-registration indicated the landing zone, and second stent size was implanted (XIENCE™ 3.5x28mm) to cover the proximal LAD.

PCI Optimization

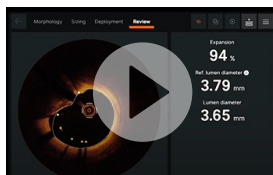

Post-PCI OCT assessment using the MLD-MAX algorithm to check for medial dissection, expansion and malapposition of the stent struts.

Post-PCI Functional Assessment

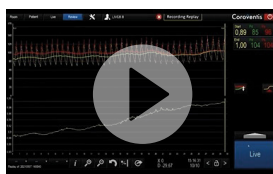

FFR pullback to assess final pressure tracing over LAD Post-PCI.

ABBOTT PORTFOLIO FOR OPTIMAL TREATMENT WORKFLOW




CORONARY GUIDE WIRES can be relied on to handle a wide variety of everyday needs and provide you with a solution for simple to complex and challenging cases.



WEBSITE



PRODUCT MATERIAL

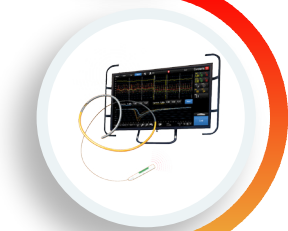

The **COROFLOW[®] CARDIOVASCULAR SYSTEM** is capable of calculating and displaying hemodynamic measurements in both epicardial vessels and coronary microvasculature.



WEBSITE



PRODUCT MATERIAL



The innovative **PRESSUREWIRE[™] X GUIDEWIRE** – the world’s only wireless physiology wire – can measure pressure and temperature to calculate Abbott’s Resting Full-Cycle Ratio (RFR), Fractional Flow Reserve (FFR), Index of Microvascular Resistance (IMR), and Coronary Flow Reserve (CFR).



WEBSITE



PRODUCT MATERIAL



The **ULTREON[™] 1.0 SOFTWARE** with an intuitive interface offers physicians a user-friendly on-screen information and step-by-step guidance following **MLD MAX WORKFLOW** to aid with decision-making to determine a proper treatment strategy pre-PCI and to ensure optimal stent expansion results post-PCI.



ULTREON[™] WEBSITE



MLD MAX WEBSITE



PRODUCT MATERIAL




DRAGONFLY OPSTAR[™] IMAGING CATHETER is a new generation of Abbott’s Dragonfly[™] Catheters used for intravascular imaging with Optical Coherence Tomography (OCT).



WEBSITE



PRODUCT MATERIAL

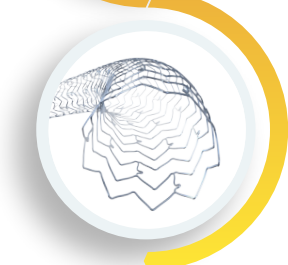

XIENCE[™] DRUG ELUTING STENT SYSTEM is uniquely designed for exceptional performance and to treat tapered lesions in small or large vessels using a single stent.



WEBSITE



PRODUCT MATERIAL

CAUTION: This product is intended for use by or under the direction of a physician. Prior to use, reference the Instructions for Use, inside the product carton (when available) or at vascular.efu.abbott or at medical.abbott/manuals for more detailed information on Indications, Contraindications, Warnings, Precautions and Adverse Events. This material is intended for use with healthcare professionals only.

Information contained herein for **DISTRIBUTION in Europe, Middle East and Africa ONLY**.
Check the regulatory status of the device in areas where CE marking is not the regulation in force.

Illustrations are artist’s representations only and should not be considered as engineering drawings or photographs. Photos on file at Abbott.

Abbott International BVBA

Park Lane, Culliganlaan 2B, 1831 Diegem, Belgium, Tel: 32.2.714.14.11

[™] Indicates a trademark of the Abbott Group of Companies.

[†] Indicates a third-party trademark, which is property of its respective owner.

www.cardiovascular.abbott

©2022 Abbott. All rights reserved. MAT-2200784 v1.0

